

## ABC BOX

*With our ABC Box, mixed up parts, will be a thing of the past!*

Intended use for the OTA "ABC" Autoclave Box for Components

1. The small compartments on the left and right are for abutments, screws, and crowns.
2. The large compartment is for bars, bridges, and larger instruments used during procedures.
3. Always secure the ABC Box with the latch on the handle.
4. The ABC Box should be placed in the Ultrasonic cleaner prior to steaming or dry sterilization.
5. Sterilization is recommended up to 338 degrees Fahrenheit.
6. Always place the Autoclave box in the sterilizer on a flat, plain surface far from the walls of the unit.
7. Never place on top of other objects.

