Exclusivly Distributed By:
Zimmer Dental Inc.
1900 Aston Ave.
Carlsbad, CA 92008 / USA
Phone: +1.800.854.7019

Medical Device and QA Services
76 Stockport Road
Timperley, Cheshire, UK WA157SN
Phone: +44 161 711 7181

Sinus Lift Balloon

Before using this product, the operating surgeon/practitioner in charge should carefully study the indications, contraindications, recommendations, warnings and instructions, as well as all other product-specific information (technical product description, description of the surgical and restorative technique, catalogue sheet, etc.) and fully comply with them. Detailed instructions over and above those contained in this instruction for use concerning the possible combinations, product-specific risks, preparatory steps, indications and contraindications, etc. can be found in the description of the surgical technique, in the technical description of the product or on the appropriate catalogue sheet. It is also recommended to attend the appropriate user-training courses. The aforementioned documents and details of the training courses may be obtained from the appropriate representatives in the various countries. Neither Zimmer, nor the manufacturer, importer or supplier of this product shall be liable for complications, other negative effects or damages that might occur for reasons such as incorrect indications or surgical technique, unsuitable choice of material or handling thereof, unsuitable use or handling of the instruments, asepsis and so on. The operating surgeon is responsible for any such complications or other consequences. It is also the operating surgeon’s responsibility to properly instruct and inform the patient on the functions, handling and necessary care of the product and on all known product risks.

DESCRIPTION:

Contains one of the following configurations:

- (1) 14002 STRAIGHT or
- (1) 14003 ANGLED or
- (1) 14004 MICRO-MINI and
- (1) 5cc sterile syringe

Key Features of the Sinus Lift Balloons:

- 14002 Straight & 14003 Angled have a 3.5mm diameter Shaft.
- 14004 Micro-Mini has a 1.9mm diameter Shaft.

INTENDED USE:

To elevate the subantral sinus membrane by balloon inflation for packing graft material into the osteotomy in preparation of the site for dental implant placement.

CONTRAINDICATIONS:

The sinus elevation procedure should not be performed if the patient has acute or chronic sinusitis or a latex allergy.

WARNINGS:

DO NOT AUTOCLAVE

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

PRECAUTIONS:

This product contains natural rubber latex which may cause allergic reactions. The patient must be informed of this prior to performing the procedure. Do not use on patients with allergic reactions to natural rubber latex.

When preparing the osteotomy, the clinician should smooth out any sharp edges of the bone before inserting the balloon.

If extensive perforation of membrane occurs during preparation of the osteotomy the balloon should be used to help seat the repairing membrane before filling the site with graft material.

Avoid storage or prolonged exposure to temperature extremes.

ADVERSE EFFECTS:

The following complications may occur relative to sinus lift elevation procedure: perforation of the membrane, acute or chronic sinusitis, allergic reaction to latex. Other adverse effects may also occur as a result of iatrogenic factors and host responses.

STERILITY:

Sterilized by gamma radiation. Refer to the packaging for verification of sterility.

Single Use

Reuse of a single use device that has come in contact with blood, bone, tissue or other body fluids may lead to patient or user injury. Possible risks associated with reuse of a single use device include, but are not limited to, mechanical failure and transmission of infectious agents.

Shelf Life

The product expiration date is indicated by the hourglass symbol on the product label, followed by the year and month of expiration. **Caution:** Do not use sterile devices if the packaging providing the sterile barrier has been damaged or compromised in any manner (i.e. opened or perforated).

TECHNIQUE INFORMATION:

Upon opening of the Sinus Lift Balloon, follow the procedures outlined below:

EC	REP	Medical Device and QA Services <p>76 Stockport Road Timperley, Cheshire, UK WA157SN Phone: +44 161 711 7181</p>
-----------	------------	--

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Medical Device and QA Services
76 Stockport Road
Timperley, Cheshire, UK WA157SN
Teléfono: +44 161 711 7181

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