

PP-1, Guide Pins For Replace Select System

This device is indicated to help the Surgeon to keep parallelism between the implants.

The Surgeon should insert soon in the surgical site after the initial perforation with the 2.0mm drill, using the inferior part of PP-1 that possesses 1.8mm diameter.

With the pin in position, he will have a great reference to maintain the parallelism in the next perforations.

The conical portion should be inserted after the end of the surgery. With the finest portion outside, he will also have a reference to conclude the end of the other surgical site.

OTA PP-1 Guide pins are supplied non sterile and should be sterilized prior to use.

NOTE: PP-1 can be sterilized using standard autoclaving procedures.