

## **PERIOTOME**



The OTA400 Periostomes come in four different blade configurations. We refer to them by Catalog numbers, Gray-OTA302, Green-OTA350, Blue-OTA301 or Gold-OTA303.

The OTA300 set of 3 consists of a Gray, Blue and Gold Periostomes. The OTA400 set of 4 consists of a Gray, Green, Blue and Gold.

The apical end of the Periostomes can be used for tapping.

Insert Periostome as far as possible into the periodontal space applying enough pressure to sever the periodontal ligaments. Separate the periodontal fibers along the surface of the root using up and down movements while GENTLY moving the blade around the root. Extract the tooth carefully to avoid damaging the facial cortical plate.

### **DO NOT USE PERIOTOMES TO REMOVE TOOTH. This will cause the blade to BREAK!**

Yellow and Blue Periostomes are indicated for buccal and lingual side of tooth

Gray and Green are indicated for the mesial and distal side of tooth.

**The Periostomes are provided non-sterile.**

**All Periostomes must be sterilized prior to clinical use using standard autoclave procedures.**

**Periostomes are intended for single use only however, they generally last between 15 and 20 extractions before breaking.**