

TORQUE ADAPTER

CATALOG NUMBER: TA15/20/25/35/45

- Proper maintenance of the torque adapter is crucial for long term success.
- Do not exceed 132°C (275°F) when sterilizing.
- In order to prevent discoloration and/or damage to the plating of the Torque Adapter from chemicals that are not sufficiently cleaned from other instruments, do not autoclave with other instruments.

Autoclave Instructions

- Disassemble the torque adapter from the bottom portion of the Driver by pulling out from the bottom section.
- Wipe the torque adapter clean.
- Wipe any debris off with an alcohol-soaked cloth.
- Insert the torque adapter into a sterilizing pouch and seal it.
- Autoclave for 15 minutes at 132° C (275° F). Do not set the sterilizer temperature above 132° C (275° F).

The Torque Adapter is expected to hold torque settings for 15-20 autoclave cycles before having to be recalibrated. If you find that your torque adapter is not working properly, you may return to us for re-calibration. A minimal charge for this service may be required.