

## **ZM Sinus Lift Kit**

The new ZM Sinus Lift Kit has 7 instruments.

# 1, is designed to reflect the Distal and Mesial portion of the Shneiderian Membrane. The 45° angle is perfect to achieve that, but in case you need a more acute angle, 90° for example, then the perfect instrument is the # 2. With both instruments, you can use in a large (long tip) or narrow sinus (short tip).

# 2 and # 4, are designed to reflect the Upper and Lower part of your sinus window. Again, the # 2 is for a large sinus and # 4 for a small sinus

# 5 is to carry and condense your bone substitute.

# 6 is also to condense your bone substitute, for those who prefer a small or a ball condenser.

# 7 is a depth marker for you check the depth of your implant placed into the sinus. One side has 2mm diameter – to be checked right after the first drill, and on the other side has 3 mm diameter. The laser marks have 8, 10, 11.5, 13 and 15mm depth.